BISI-Tray

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BISI-Tray

Product Description

Bisi-Tray is a pink, light curing tray material in form of prefabricated, upper base plates to make rigid, custom impression trays and bite-blocks, as well as for other indications (refer to "Indications")

Bisi-Tray can be cured in all ultraviolet and Halogen light curing units.

The base plates are supplied in a re-usable light-proof box which can be used for transporting or storing models etc.

Indications

Custom impression tray, Bite-block, Base plate for setting up dentures, Articulator guide pin excursion tracing, Bite wafer for trying in framework, Splinting outer coping or telescoping crowns

Preparation

Mark the outline of the future tray on the model.

Block out the undercuts on the model.

To create a spacer for the impression material, block out the entire model (e.g. with a sheet of wax) and cut stops out.

If the plaster is very dry, it is recommended to soak the model briefly with water or apply an alginate based isolation .

Application

Remove the **Bisi-Tray** base plate from the box and close it.

Once the model has been blocked out, adapt the base plate to it and cut to shape using, e.g. a scalpel.

Use the remaining material to form a handle and attach it to the tray.

Place the model and tray in a light curing unit and cure without vacuum.

Remove the tray from the model and polymerize its underside.

If the tray has been cured in air, use alcohol to remove the smear layer.

When using a **HIGHLIGHT ST** light curing unit, the second curing cycle should be run with programme 2 (in vacuum) to eliminate the smear layer.

Times

Working at daylight 20 minutes

Polymerisation in a **HIGHLIGHT ST**:

1. On the model, in air (programme 3) 7 minutes
2. Off the model, in vacuum (programme 2) 2 minutes

Polymerisation in an **ultraviolet or Halogen light curing unit**, depending on the unit in use:

1. The top of the tray, on the model 3-5 minutes
2. The underside of the tray, off the model 3-5 minutes

Trimming

Always use a dust extractor when trimming the tray!

Trim the custom tray with tungsten carbide burs and smooth it with sandpaper. Pumice slurry can be used for polishing.

Hygiene

Immerse impressions taken with **Bisi-Tray** custom trays in a disinfectant solution based on glutaraldehyde (can be cleaned with EXTRACLEAN or an The time depends on the disinfectant manufacturer's directions.

Incompatibility

Bisi-Tray may cause sensitization by sensitive persons. If such reactions are experienced, discontinue the use of **Bisi-Tray**.

Technical Data

Flexural strength 120 N/mm²

Storage

Do not store **Bisi-Tray** above 25 C / 77°F.

Protect against light.

Shelf-life

Do not use after the expiry date.

Federal law restricts this device to sale by or on the order of a dentist.

Bisi-Tray pink: **REF** 03250

Bisi-Tray blue: **REF** 03270

Made in Germany

Rev.: 65900-05-2018