## PROVI BOND K

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# *bisico*

## PROVI BOND K

**Non-Eugenol Temporary Cement in cartridges for Cementing Crowns and Bridges**

**Provi Bond K** is an **non-eugenol temporary crown and bridge cement**, cartridges and mini syringes, suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. It shows excellent adhesion to temporary crown and bridge materials, but it can easily be removed from prepared tooth stumps. **Provi Bond K** comes in cartridges and syringes, so it is very easy, comfortable and material saving to use. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on ZnO/organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

**Provi Bond K** meets the requirements of **DIN EN ISO 3107** and the **ADA/ANSI Specification No. 30**.

### APPLICATION

**No dispensing gun is needed for using the 8,5 g mini-syringe.**

### Application gun and cartridge

Remove and dispose the closure-cap. (don't use it again) Install the attached mixing tip onto the cartridge or syringe and fix it with a quarter turn to the right. The material is now ready for application. After each use leave the used mixing cannula in place to seal the cartridge.

### Note

Discard the first 2-3 mm of the extruded material. (about the size of a pea) Then the following mix will be perfect. This must be done for each new mixing tip.

### Cementing

**Provi Bond K** can be applied directly into the temporary crown or bridge under slight pressure from the mixing cannula.

Use a thin layer to the inside of the temporary crown or bridge and seat not later than **1:30 minutes** (working time) cautiously but firmly over the prepared tooth structures.

### Note:

Do not dry stumps, but rather moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

**Provi Bond K** sets in the mouth within **2-3 minutes**. After **Provi Bond K** has set remove any excess.

### Precautions

- Do not use **Provi Bond K** in cases where known allergies to the components exist.
- In case of contact with skin and eyes, rinse immediately with plenty of water.
- If swallowed advance consult a physician.

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BisicoBielefelder Dentsilicone GmbH & Co. KG, Johanneswerkstr. 3, 33611Bielefeld, Tel. 0521-8016800